

# AEOLUS


## Professional Vet Equipments Supplier




AEOLUS

## Grooming / Operation Table with LED Light


FT-829

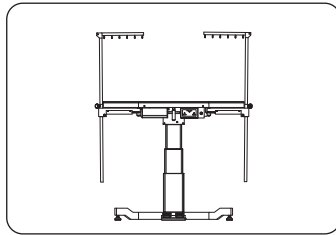


FT-889

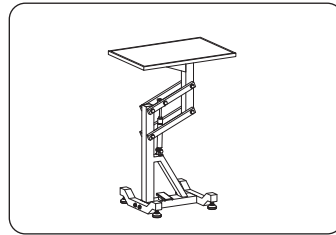

FT-838

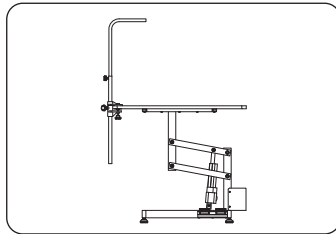

FT-802

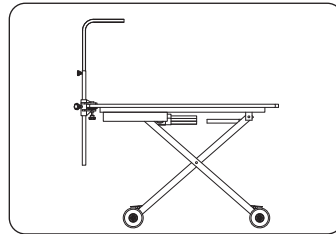

FT-833E

Distributor Contacts


### **DALLAS**

11307 Indian Trail, Dallas, Texas 75229, USA.  
Tel: +1 469 930 6359

### **SHANGHAI**


Floor 5, Building 6, No. 675, Zhujin Road, Songjiang District, Shanghai, China.  
Tel: +86-21-67632060, 67632364

New  
Generation

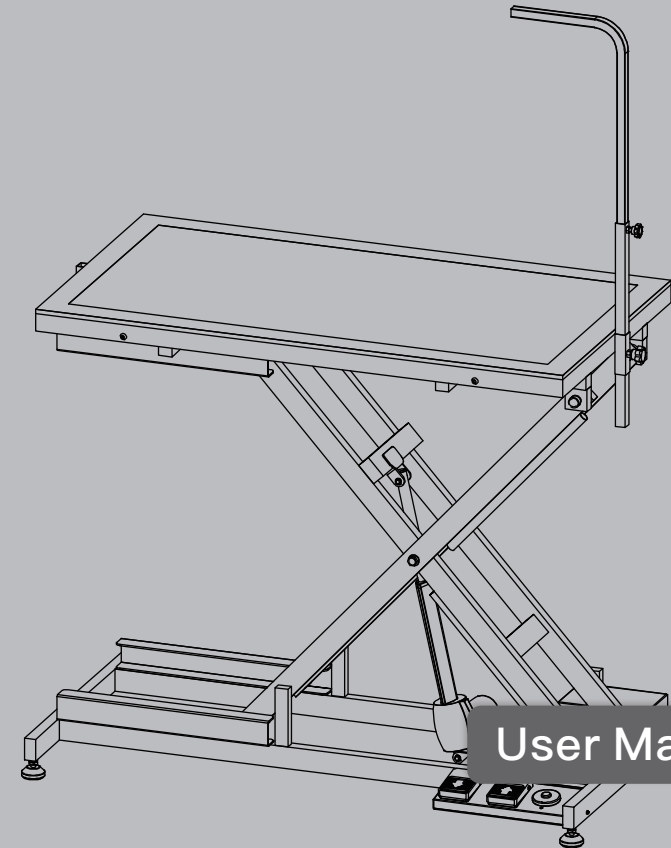

User Manual

# Overview

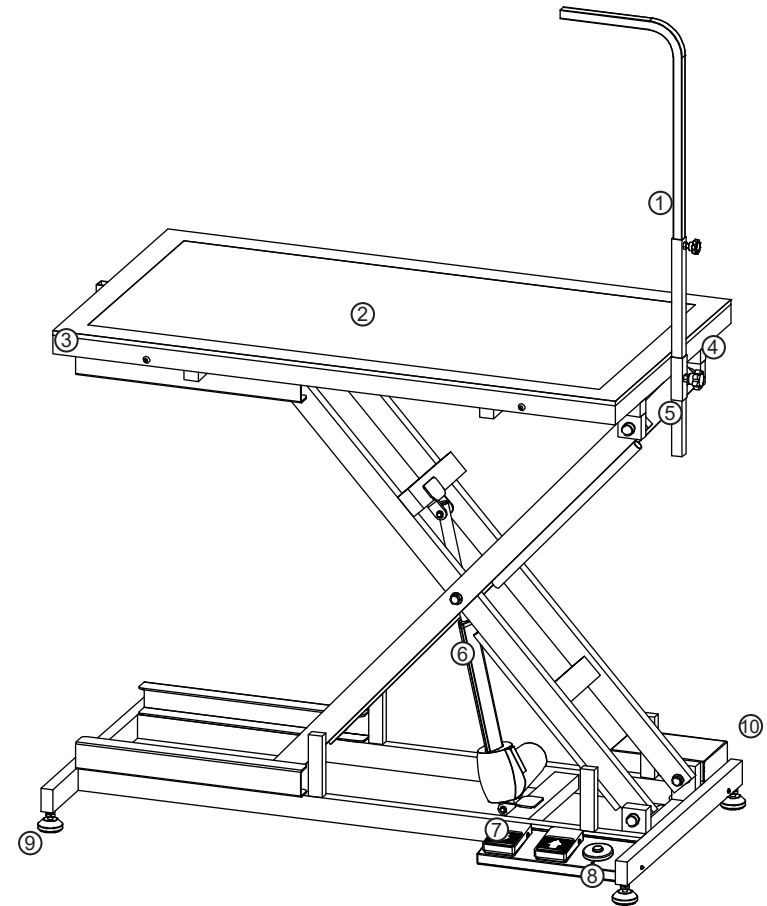

- ① Grooming Arm
- ② LED Lighting Tabletop
- ③ Stainless Steel Frame
- ④ Grooming Arm Mount
- ⑤ Tightening Knob
- ⑥ Electric Lifting Base
- ⑦ Foot Control Pedals
- ⑧ LED Light Switch
- ⑨ Adjustable Levelers
- ⑩ Power Cable and Plug

# Specification

Dimension	120×60×25.5—95cm 47-16/64×23-40/64×10-3/64—37-26/64inch
Package	132×71×44cm 51-62/64×27-61/64×17-21/64inch
Rated Voltage	AC 220V 50Hz
Net Weight	75.5kg
Gross Weight	96kg
Loading Capacity	200kg

**AEOLUS Electric Lifting Table FT-829**, rock-solid at any height (255mm—950mm), features an electric lifting base that is easily controlled by water-resistant foot pedals, a tabletop with anti-slip and scratch resistant, mat finish tempered glass surface, that can be lighted by a LED cold light panel below, a stainless steel top frame that is durable and long lasting, and a skid-proof silicon mat that does not affect or diffuse the light below.

# Package Inspection

The manufacturer is not liable for any damage caused by a third party "Transport or Delivery Company".

The day the unit is delivered, an inspection of the outer packing should be made for any damage to the packing box.

Then the unit should be unpacked carefully; all components, accessories and the table itself should be inspected for any damage or loss.

If any damage is evident, the carrier or dealer should be notified immediately.

# Safety Warning

1. Use only as described in this manual. Use only manufacturer's recommended attachments.
2. To reduce the risk of fire, or electric shock – DO NOT use outdoors or on wet surfaces. DO NOT place the product on an uneven or soft ground.
3. This product is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the product by a person responsible for their safety. Children should be supervised to ensure that they do not play with the product.
4. DO NOT use with damaged cord or plug.
5. DO NOT cut, damage, or modify the cord. DO NOT pull or carry by cord, use cord as a handle,

close door on cord, or pull cord around sharp edges or corners. DO NOT pull, twist, or bend the cord with unnecessary force. DO NOT run the product over cord. Keep cord away from heated surfaces. DO NOT place heavy objects on the cord or allow the cord to get pinched in between objects. DO NOT wind the cord around the product.

6. DO NOT unplug by pulling on cord. To unplug, grasp the plug, not the cord.
7. DO NOT handle plug or the product with wet hands.
8. Turn off all controls before unplugging.
9. DO NOT use the product near flammable or combustible liquids such as gasoline or use in areas where they may be present. The fumes from these substances can create a fire hazard or explosion.
10. DO NOT modify, disassemble or repair.
11. Make sure the local voltage and frequency of the power meet the requirements on the label of the product.
12. Avoid using the product in dusty, vibrating and wet environment and in chemical storage place.
13. Neither people nor animals should be under the tabletop when it is power-on.
14. Gently control the tub to rise or lower slowly and safely.

# Assembly Instructions

Insert the grooming arm into the grooming arm mount and use the tightening knob to fix the grooming arm.

# Operation Instructions

1. Plug in.
2. Step on the up/down foot pedals to adjust the height of the table as needed.
3. If needed, step on the LED light switch to light up the tabletop.
4. Put the animal on the tabletop and start working on it.
5. After work is completed, remove the animal, switch off the light and unplug the table.

# Parts Replacement

## ***Electric motor replacement***

1. Disconnect the powder cord from the wall socket. Remove the tabletop from the frame assembly, and set it aside. Remove the grooming arm.
2. Disconnect the electric motor cable from the motor control unit. Remove the snap rings and the retaining bolts that hold the electric motor to the frame assembly. (See Figure 1 and 2)

- Using help, lift the frame assembly up, and remove the entire motor assembly from the frame.
- Install the new motor assembly in the reverse order. Reconnect the cable to the motor control unit.
- Plug the power cord into a wall socket and test the table for proper operation.

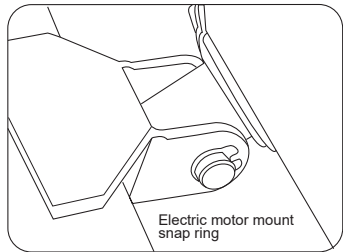


Figure 1

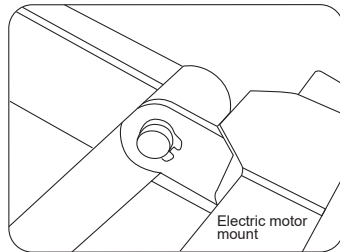


Figure 2

### Control pedal replacement

- Disconnect the power cord from the wall socket.
- Remove the three screws holding the control pedal to the frame assembly.
- Disconnect the control cable connector from the motor control unit. (See Figure 3)
- Install a new pedal in the reverse order, and re-connect the cable to the motor control unit.
- Plug the power cord into a wall socket and test the table for proper operation.

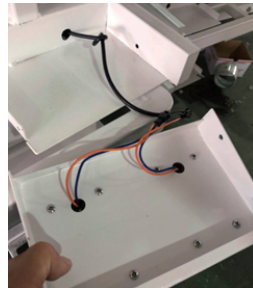


Figure 3

## Maintenance

In between use, the table should be cleaned with a soft cloth or sponge using a mild cleaning agent.

## Warranty

Each Aeolus product has a 1 year limited warranty. This warranty covers the cost of parts and repairs done by Aeolus International's Technician Team. Your warranty is only valid when accompanied by your sales receipt or proof of purchase date. This warranty coverage excludes lost items or destroyed items. This warranty does not cover the shipping costs associated with your repair or replacement items. To start a product repair or parts replacement claim for your product, please contact us at [info@AEOLUSpet.com](mailto:info@AEOLUSpet.com) or call your local Aeolus location.

## Service Support

Contact us if you cannot solve your problem with this manual.

### Dallas

11307 Indian Trail, Dallas, Texas 75229, USA.

Tel: +1 469 930 6359

Web: [www.aeoluspet.com](http://www.aeoluspet.com)

### Sydney

147 Dwyer Road, Leppington NSW 2179, Australia

Tel: +61 2 9606 4520

### Singapore

150 South Bridge Road, Singapore 058727

Tel: + 65-6533 6524 / 9633 2734

### Shanghai

Floor 5, Building 6, No.675, Zhujin Road, Songjiang District, Shanghai, China

Tel: +86-21-67632060, 67632364